



FACE MASKS & COVID-19

MASKS AND CORONAVIRUS TRANSMISSION

Masks help to control the spread of the coronavirus by capturing respiratory droplets before they enter the airstream. They work best when EVERYONE is masked.

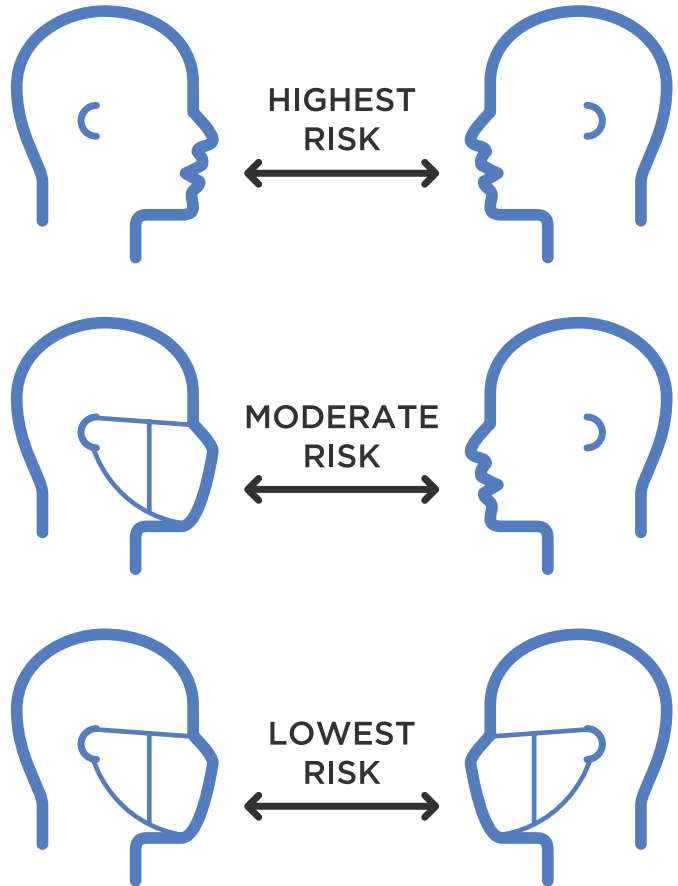

WEAR YOUR MASK CORRECTLY

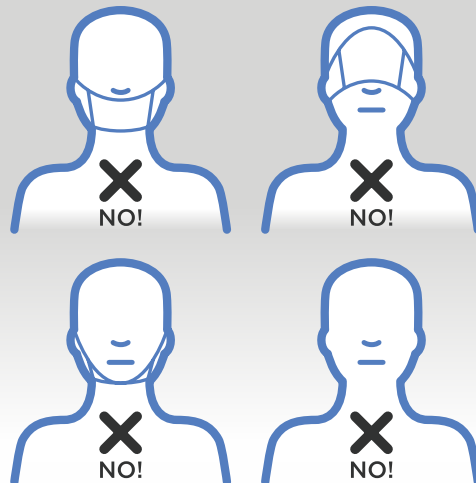


CHOOSE THE RIGHT MASK

Different types and styles of masks have different levels of effectiveness. Choose the mask that is right for your level of exposure and work environment.




	Cloth Masks	Surgical Masks	N-95 Respirators
Look for:	<ul style="list-style-type: none"> Multiple layers of fabric Tight weave Disposable filter pocket (optional) 	<ul style="list-style-type: none"> ASTM rating 2 or 3 (for fluid protection) 3 layers of protection Nose bridge strip for better fit 	<ul style="list-style-type: none"> NIOSH Approval Tight seal against face (must be fit tested)
Best for:	<ul style="list-style-type: none"> General public Daily use in non-medical/low-risk environments 	<ul style="list-style-type: none"> Commercial and industrial environments Low- and moderate-risk medical environments (no direct contact with known COVID patients) 	<ul style="list-style-type: none"> High-risk medical environments/COVID patient care Industrial environments needing both small particulate and pathogen protection
Protect Against:	<ul style="list-style-type: none"> Large respiratory droplets (Good) Small respiratory droplets (Moderate) Large particulates (Moderate) Small Particulates (Poor) Fluid spray (none) 	<ul style="list-style-type: none"> Large respiratory droplets (Excellent) Small respiratory droplets (Good) Large particulates (Good) Small particulates (Good) Fluid spray (Excellent) 	<ul style="list-style-type: none"> Large respiratory droplets (Excellent) Small respiratory droplets (Excellent) Large particulates (Excellent) Small particulates (Excellent) Fluid spray (Moderate)
Features:	<ul style="list-style-type: none"> Washable 	<ul style="list-style-type: none"> Disposable Fluid Protection 	<ul style="list-style-type: none"> Disposable Fit tested
Protection Type:	<ul style="list-style-type: none"> Protects others from wearer 	<ul style="list-style-type: none"> Protects others from wearer Provides some protection for wearer 	<ul style="list-style-type: none"> Protects others from wearer Protects wearer

DO NOT USE:

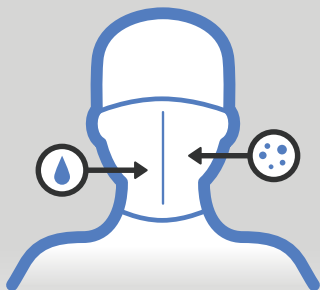


T-SHIRT OR LOOSE WEAVE SYNTHETIC CLOTH MASKS

- Does not contain respiratory droplets
- May aerosolize droplets, increasing airborne spread

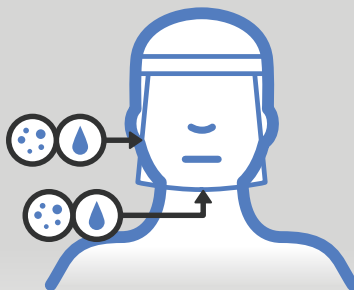

FACE SHIELDS

- Does not contain respiratory droplets
- Should only be used for additional fluid spray protection WITH a mask

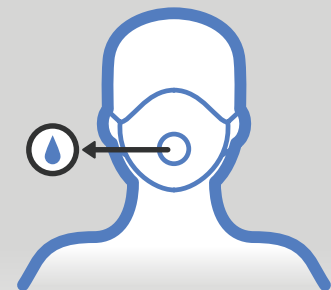

DUST MASK WITH EXHALATION VALVE

- Does not contain respiratory droplets

RENSA
HEALTH

RENSAHEALTH.COM

Sources: CDC; Health Grades